C | S

47102
MITELLA
WRAP

47102 ARMSLINGA MED OMTAG

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47102**MITELLA WITH WRAP**

Carefully read this instruction sheet before use!

ENGLISH

Intended use/Indications: Intended for temporary off-loading for shoulder and/or arm.

Indications: Post-trauma and post-surgery conditions.

Contra indications: None known.

Features:

This product consists of two parts that can be adjusted independently of each other.

Adjustments:

This product should only be altered by a medical professional.

Washing Instruction:

Close all Velcro prior to laundering. Machine wash in 60°C (140°F). Hang to dry. Do not tumble dry or dry clean.

Material Specifications:

Main material: 100% Polyester, foam made from Polyurethane.

Velcro: 100% Nylon

Buckle: 100% POM

Straps and Edging tape: 100% Polyester

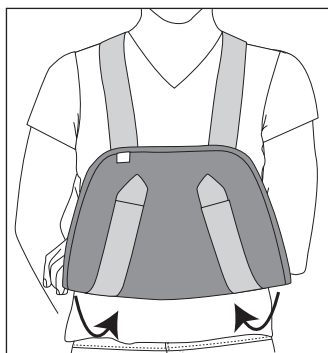
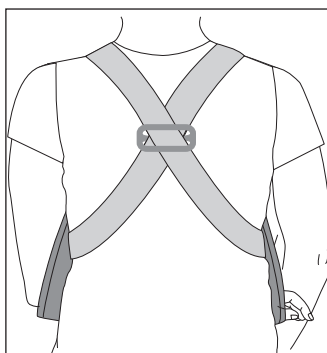
This product is not flame retardant.

Who to Contact:

If any problems occur, remove the support and contact your fitter or medical practitioner.

Time of use:

Follow the medical practitioners guidelines for use and limitations. This product is recommended for single person use only.

Instructions for use

1. Check that the product is complete and without damage.
2. Loosen the straps so they can be easily be placed over head.
3. Take the straps from behind, pull them forward and attach to the front as shown.
4. Put the wrap around the body and the injured arm and attach.
5. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38