**EpiGel**  
28166

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Carefully read these instructions before use!

### Intended use/Indications

Lateral and / or medial epicondylitis (so-called tennis and golf elbow).

This product is intended for mild and temporary complaints. The bandage relieves the muscle attachment by a slight pressure.

### Contraindications

Edema, swelling.

### Adjustments

This product should only be altered by medical professional person.

### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

### Material specification

Nylon, Polymer Gel, Polyester

### Wash instruction

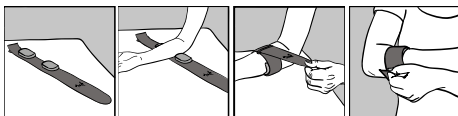
Attach all Velcro straps to the fabric and remove the gel pads prior to laundering. Machine wash on gentle wash 40°C with a mild detergent without bleach. Dry flat. The pads can be wiped off with a damp cloth.

### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

## Instructions

1. Check that the orthosis is complete and without damage.
2. Put it flat on a flat surface with the inside upwards (completely open).
3. Position the arms widest part on the orthosis approx. 5 cm below the elbow (palm of hand facing upwards) between the pads. Position the removable pads just below the point of pain.
4. Relax the muscles and attach the strap. Check the pads position according to above. Check that the orthosis is firmly strapped without interfering with blood circulation.
5. Move the wrist back and forth and feel that the orthosis moves with the muscles.
6. Adjust for comfort.

Med Spec Item No.	Size	CM
223410	One size	20-35 cm

Medical Specialties, Inc.  
4600-K Lebanon Rd.  
Charlotte, NC 28227  
p: 800-334-4143 f: 800-694-9060  
p: 704-694-2434 f: 704-694-9060  
e-mail: request@medspec.com  
www.medspec.com

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MDSS GmbH
Camp Scandinavia AB, Karbingatan 38