**OPED GmbH**

Medizinpark 1
83626 Valley/Oberlaindern
Germany
Fon +49 (0)8024/6081 8-210
Fax +49 (0)8024/6081 8-299
mail@oped.de
www.oped.de

OPED AG

Hinterbergstrasse 26
6312 Steinhausen
Switzerland
Fon +41 (0)41/748 53 35
contact@oped.ch
www.oped.ch

OPED UK Ltd.

Unit 5
Avro Business Centre
Avro Way
Bowerhill
Melksham
SN12 6TP
United Kingdom
Phone +44 (0)1380 722177
enquiries@oped.biz
www.oped-uk.com

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VACOpedes

Please read the instructions for use and safety advice before using VACOped. You will find a readable instruction manual on our OPED homepage.

Purpose: Orthosis designed to immobilise joints in the specified position and provide functional treatment for mid-foot and forefoot injuries.

Safety information

-  The product is intended only for use on a single patient. 
-  VACOpedes is not designed for dynamic stresses such as jumping and fast walking. If you fall with the VACOpedes fitted, check your VACOpedes immediately for damage. If the VACOpedes is damaged, it can impair your safety.
-  Never wear the product over open wounds and only wear it after prior medical instruction.
-  The patient may only change the position of the device after consultation with the physician responsible for their care.
-  If excessive pain, numbness or sensation changes occur, or if unusual reactions or an unpleasant feeling are experienced while wearing the device, please contact your doctor or orthopaedic technician without delay.
-  This medical product has been developed for the indications listed and can be damaged by unintended or inappropriate use, by activities such as sport, falls or other accidents, and by liquids that are not compatible with plastics.
-  When used in combination with another medical product, consult your doctor.
-  Only spare and/or replacement parts approved by OPED GmbH should be used.
-  Please note the Road Traffic Act applicable in your country.
-  Note: Not made from natural rubber latex. 
-  Check that the sole fits securely - otherwise there is a risk of slipping!
Sole: When the toe section is fully extended, the gait may be modified or restricted.
-  The small spheres in the vacuum cushion can slide around. We therefore recommend opening the valve every two to three days, smoothing any collections of small spheres out and then restoring the vacuum.
-  Storage: The valve flap must always be closed!
The valve ring must be pushed downwards and the valve flap closed (Figure 1 + Figure 2).
-  This product is intended to be used for no more than 4 months.
-  Note: When wearing the orthosis, make sure that the shoes are high enough on the opposite side or that the sole is raised to compensate for any difference in height.

How to use

Fitting the VACOpedes

Pull the valve ring upwards (details can be found in "Valve functions") to allow air to flow in (Figure 3). If the small spheres in the liner (Figure 4) have grouped together, stroke them smooth. Place your foot in the fabric liner (Figure 4) and position it with the fabric liner all the way back in the lower shell (Figure 5). Then close the fabric liner (Figure 6). Pull the liner slightly upwards (Figure 7). Fit the upper shell (Figure 8). Slide the tabs on the upper shell between the lower shell and the liner. Close the straps by hand and as the pain allows (see graphic for sequence) (Figure 10). Retighten the straps again after a few steps. When fitting the upper shell, ensure that the valve is not kinked. The upper shell should not be positioned against or over the valve. If the physician permits, apply some pressure on the foot briefly so that the foot is moulded in the cushion. Then create the vacuum.

Adjusting and securing the vacuum cushion

Readjustable vacuum cushion: Inside the liner, there is a cushion containing thousands of small spheres (Figure 11) which adapt perfectly to the anatomy and any swelling of your foot. Pressure-sensitive areas and swellings can be relieved through the targeted redistribution of these small spheres. To ensure that the spheres do not slide around inside the cushion, the air is extracted via the valve using a suction pump, creating a vacuum (Figure 12).

Valve functions

Open the valve / softens the vacuum cushion:

Pull the valve ring upwards and move it to the side to allow air to flow in (Figure 3). Then slide the valve ring back to its original position.

Close the vacuum / hardens the vacuum cushion:

Push the valve ring downwards. Attach the suction pump and squeeze it (approx. 7-8 times) until it no longer inflates. Squeezing the pump removes air from the cushion and creates a vacuum. This fixes the adapted fit in place (Figure 13). To finish, close the valve cap (Figure 14).

Closing the valve cap

Push the valve ring downwards and close the valve cap. This sets the vacuum (Figure 14).

Changing the cushion liner

Note: The replacement liner does not contain a cushion! When changing the liner, the cushion must be removed from the used liner and inserted into the new one!
Open the rear zip fastener all the way and remove the vacuum cushion from the used liner. Open the valve and shake out the cushion so that the small spheres can redistribute themselves (see Valve functions). Establish the vacuum (see Valve functions), place the vacuum cushion in the new liner and slide the valve through the button hole (Figure 11).

A second cover is not included in the box in all countries. If the supplied package does not include a second cover, this must be ordered separately from OPED.

VACOpedes

Fitting and changing the sole

Removing the sole

Push both unlocking buttons simultaneously (Figure 15).

Fitting the sole

Position the VACOpedes straight and centred on the sole before pushing it against the sole (you will hear two clicks) (Figure 16).

Adjusting the sole length

Remove the sole (see Fitting the sole) and push the unlocking button. Pull the toe section to the desired length (Figure 17).

Patient information

Pressure points / problem-solving

Remove the orthosis from your foot. Pull the valve ring to allow air to flow into the cushion. Spread the cushion filling evenly at the pressure point. Re-fit the orthosis (see How to use). Using a finger, check that there are enough small spheres in the cushion between the pressure point and the outer lower shell.

Showering / washing

We recommend using a shower protector when showering. If the orthosis or its cover gets wet, they must be dried.

Sleeping

Before you go to sleep, place a towel, such as a hand towel, underneath the device so that the bed sheets are not damaged.

Product characteristics

Sizes

The product can be worn on the left or right.

Recommendation: At the size interfaces between the Small/Medium and Medium/Large versions, supply the smaller VACOpedes size if possible.

	Small	Medium	Large
EUR	35 – 40	40 – 45	45 – 50
UK	2 ½ – 6 ½	6 ½ – 10 ½	10 ½ – 14
US (M)	3 ½ – 7 ½	7 ½ – 11 ½	11 ½ – 15
US (F)	4 ½ – 8 ½	8 ½ – 12 ½	12 ½ – 16

Indications

- Metatarsal fractures I - IV
- Hallux valgus / rigidus
- Arthrodesis of the metatarsal joints
- Hammer and claw toes
- Lapidus Operation

Contraindications

None known

Information regarding care / cleaning / disinfection / sterilisation

You are receiving this medical product in a hygienically perfect condition.

Cleaning instructions for patients:

- Lower shell: Dishwasher at 60 °C
- Strap: Washing machine at 40° C
- Terry liners: Washing machine at 60 °C

Remove the vacuum cushion from the Terry liner before washing it. Close the zip and Velcro fasteners again for washing.

- Drying: Air dry, do not dry in a tumble dryer.
- Disinfection: Wipe disinfection with plastic-compatible disinfectant.
- Sterilisation: This medical product is not suitable for sterilisation.

Technical data / parameters

Materials used:

Hard lower shells (PP); Liners (cotton/PU foam); Cushion sleeve (PVC); Filler material (EPS spheres); Strap (PP fabric); Sole (EVA); Upper shell (EVA/PP); Upper shell clasp (POM).

Transport conditions / storage conditions / disposal

Storage: Store in a dry and hygienic place and protect from direct sunlight
Germany/Switzerland: The medical device is a rental product and must be returned to OPED!

Other countries: Please dispose of the medical device in residual or household waste.

Warranty

The warranty is provided in line with statutory regulations.

In the event of complaints relating to the product, such as damage or problems with the fit, please contact the manufacturer directly. Only serious events which can lead to a significant deterioration in health or death should be reported to the manufacturer and the responsible authorities of the member state. Serious events are defined in Article 2 (65) of Directive (EU) 2017/745 (MDR).

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