**46020  
FINGER  
SPLINT**

VISSION™

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### Intended Use/Indications

For immobilizing the fingers following trauma or operation and preventing contractures of the finger joints. Can also be used in conservative finger therapy and Dupuytren syndrome. Conditions requiring temporary finger immobilization.

### Contraindications

No known.

### Adjustments

This product should only be altered by medical professional person.

### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

### Material Specifications

This product is made from Polyamid, Spandex and Polyurethane. Splint is made in aluminium. This product is not flame retardant. Does not contain Latex.

### Washing Instruction




Close all Velcro prior to laundering. Remove spoons before laundering. Machine wash in 40°C (104°F) gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean. Take notice of how the splint is positioned in the pocket to ensure correct position after wash.

### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

## Instructions for use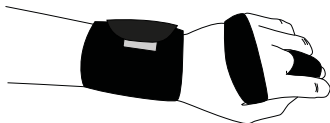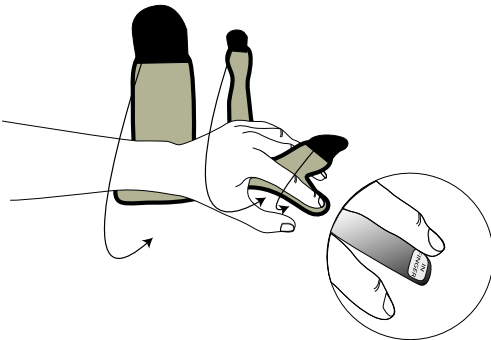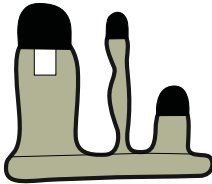

Check that the product is complete and without damage.

1. Open all Velcro straps. Product should be open and flat.
2. Position hand and fingers on the support. The treated fingers should be positioned over the long extended finger section.
3. Secure the strap around the treated fingers on the long extended finger section.
4. Wrap the wider strap around the wrist and secure.
5. Attach the middle strap around the mid-section of the palm inside of the thumb but around all fingers.
6. Adjust for comfort.

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Camp Scandinavia, August, 2015 ©
Camp Scandinavia AB, Karbingatan 38