X-LITE® CLASSIC ♦ PREMIUM ♦ PLUS ROLLS

CS

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EN

Intended use

X-LITE® is intended for construction of all common orthopaedic casts and splints. ♦ X-LITE® may only be applied by medical professionals. ♦ See packaging for best before date.

Instructions for use

Turn the rolls in the Splint bath so they are evenly heated on all sides. ♦ Use tongs to remove rolls from water. ♦ Remove water with a firm shake. ♦ Rolls with plastic separator can be squeezed into a towel. ♦ Residues of polymer can appear in the water bath and around the workstation. ♦ Remove with warm water and mild cleaner. ♦ Apply with ~50% overlap.

Precautions & Warnings

- ♦ X-LITE® is ready for full load bearing after 30 minutes.
- ♦ **Plus:** Apply the cast with the textile coated side out.
- ♦ **Classic / Premium:** For application turn the roll. The plastic separator is continuously removed during the process. Pulling the separator with one hand thereby advances the roll. Tear off the excess separator material during the casting.
- ♦ Inform the patient to avoid warm temperatures and strong sunshine.
- ♦ X-LITE® is sensitive to chlorinated water.
- ♦ X-LITE® can be used multiple times on the same patient.
- ♦ Store in a dry, dark place at temperatures between 0°C and +40°C.
- ♦ Dispose in accordance with local regulations. Can be handled with the household waste.
- ♦ Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Material specification

X-LITE® contains cotton, polycaprolactone and polyester. Latex free.

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