C | **S**

23552 NAVY-X HIGH

NAVY X™

23552 NAVY-X HÖG

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23552 NAVY-X HIGH

Carefully read this instruction sheet before use!

ENGLISH**Indications / Intended Use**

Navay-X is a lumbosacral back support to be used for light back pain and back weakness. It can also be used to support the spine during heavy lifting.

Contraindications

None found.

Adjustment

This product should only be altered by a medical professional person.

Warning

Do not pull the straps too tight, as this can cause poor circulation and/or skin irritation. If you feel any irritations or other problems, take it off immediately. Check skin on a daily basis.

Material specifications

78% Polyester, 22% Lycra, plastic stays. This product does not contain Latex. This product is not flame retardant.

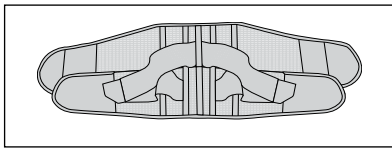
Wash instruction

Close all Velcro prior to laundering. Machine washable in 40°C (104°F). Dry flat. Do not tumble dry or dry clean.

This backsupport might have extra rigid stays added in the back section. These should be removed before laundering.


Who to contact

If any problems occur please contact your orthotist or fitter.

Instructions for use


1. Check that the product is complete and without damage.

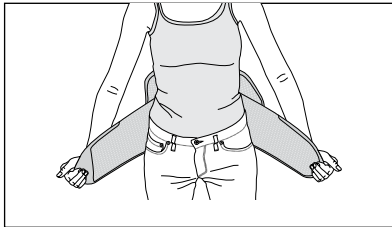2. Start by placing the stabilizing straps on the side tabs.


2. Put the brace on and close the lower front closure. Make sure it is properly positioned and centered in the lower back area.

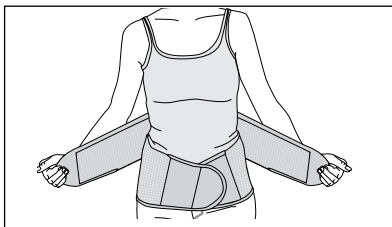3. Adjust until it feels comfortable. Then close the upper front closure on top of the lower section.



4. Take the extra stabilizing straps from the tabs and pull to front. Secure on the front Velcro.

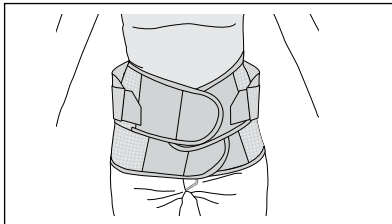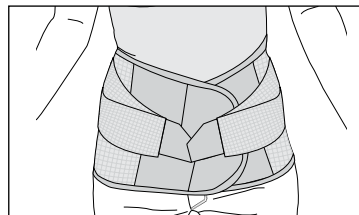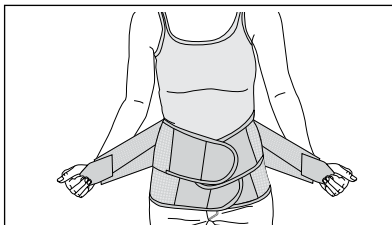5. To remove: First open the stabilizing straps and place them on the side tabs.


6. Then open the closure.

Item No.	Description	Colour	Sizes
23552	NAVY-X HIGH	Dark Blue	XS - XXL

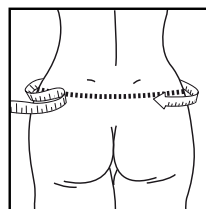Size	Hip circumference
XSmall	75 - 85 cm
Small	85 - 95 cm
Medium	95 - 105 cm
Large	105 - 115 cm
XLarge	115 - 125 cm
XXLarge	125 - 135 cm

	<p>Camp Scandinavia AB, Karbingatan 38 SE-254 67 Helsingborg, Sweden Phone +46 42 25 27 01</p>	
---	--	---